


# How to Use a Glycol Min-Max Thermometer

Always make sure the screen reads 'OUT'. If it reads 'IN', press the in/out button on the back of the thermometer.

Always make sure the screen reads "MIN" and "MAX". If not, press the "MODE" button.

Note: the "MODE" button enables the alarm setting.

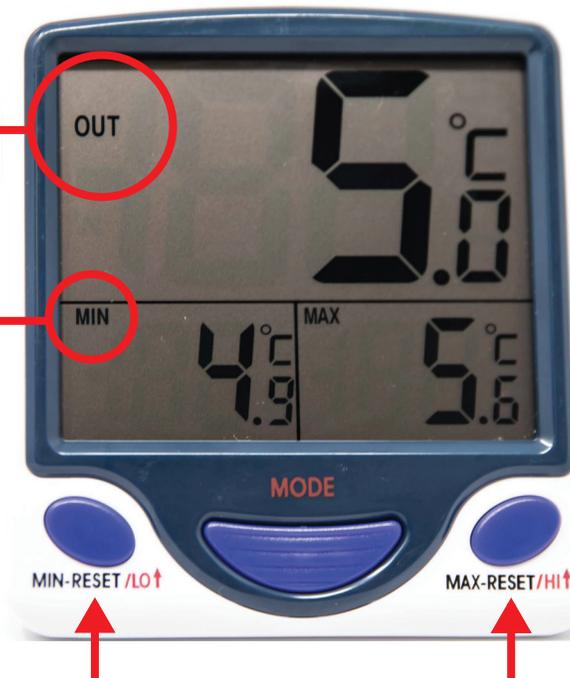

Record the minimum, maximum and current temperature in your temperature log book twice per day.

*Reminder: batteries need to be changed every six months.*

After you have recorded the temperatures, you must reset the thermometer by pressing each 'reset' button. All three temperatures should display the same number. The memory is now clear and ready to record new temperatures.

A copy of the Vaccine Storage and Handling Guidelines can be found on [health.gov.on.ca](http://health.gov.on.ca).

For more information, visit [halton.ca](http://halton.ca) or call 311.

## Safe Vaccine Storage


### Remember:

- Stock a maximum of one month supply of vaccine in the refrigerator.
- Do not leave vaccine outside the refrigerator.
- Do not open the refrigerator door unnecessarily.
- Secure the thermometer probe on the refrigerator's middle shelf inside an empty vaccine box.
- **Call 311 immediately if temperatures are outside of +2°C to +8°C.**

# How to Use a Digital Min-Max Thermometer

The main screen displays current temperature. To show minimum or maximum temperature, press the "Memory" button.

Record the minimum, maximum and current temperature in your temperature log book twice per day.

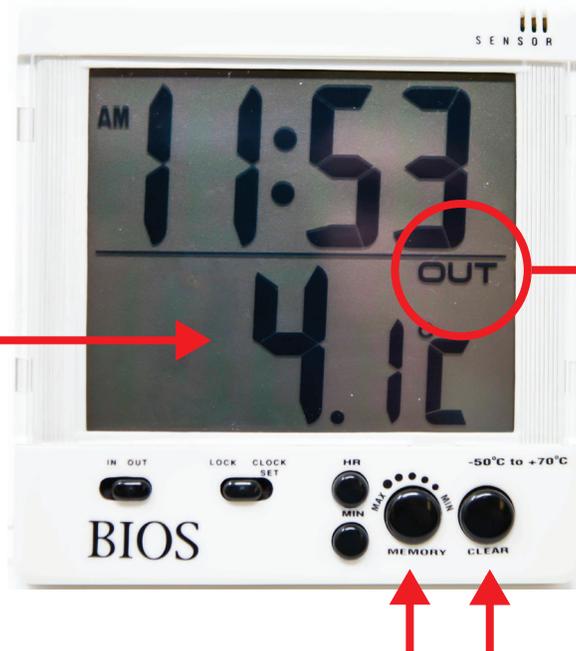

Always make sure the screen reads 'OUT'. If it reads 'IN', slide the in/out button on the bottom left hand corner of the thermometer.

*Reminder: batteries need to be changed every six months.*

After you have recorded the temperatures, you must reset the Min and Max temperatures by pressing "Memory" and "clear" then "Memory" and "clear".

A copy of the Vaccine Storage and Handling Guidelines can be found on [health.gov.on.ca](http://health.gov.on.ca).

For more information, visit [halton.ca](http://halton.ca) or call 311.

## Safe Vaccine Storage

### Remember:

- Stock a maximum of one month supply of vaccine in the refrigerator.
- Do not leave vaccine outside the refrigerator.
- Do not open the refrigerator door unnecessarily.
- Secure the thermometer probe on refrigerator's middle shelf inside an empty vaccine box.
- **Call 311 immediately if temperatures are outside of +2°C to +8°C.**